

QuikClot[®] COMBAT[®] GAUZE[®]



TraumaPad[®]

For Temporary External Use
to Control Traumatic Bleeding

ONE (1) 12 IN X 12 IN (30 CM X 30 CM) HEMOSTATIC DRESSING

NSN 6510-01-583-4870




Made in the USA


DIRECTIONS FOR USE




1. Open package and remove **TraumaPad**. Keep the empty package.

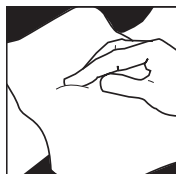

2. Place **TraumaPad** into wound and use it to apply pressure directly over bleeding source. More than one **TraumaPad** may be required.

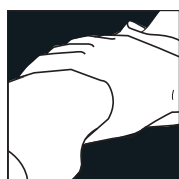

3. Continue to apply pressure for 3 minutes or until bleeding stops.

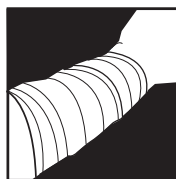

4. Wrap and tie bandage to maintain pressure. Seek medical care immediately. Show **PRODUCT REMOVAL** directions on package to medical personnel.

PRODUCT REMOVAL: 1. Gently remove gauze from wound. 2. Thoroughly irrigate wound.

Manufactured by:
Z-Medica[®], LLC
4 Fairfield Boulevard
Wallingford, CT 06492 USA
Tel: 1.877.750.0504, +1.203.294.0000
Fax: 1.800.343.8656, +1.203.303.7216
QuikClot.com
Patents: Z-Medica.com/patents.aspx
© 2020 Z-Medica, LLC
All rights reserved.
Contains kaolin.
101654-01-WEB Rev G 10/8/2020

FOR USE BY TRAINED EMERGENCY RESPONDERS



Do Not Reuse



WARNINGS: For external use only. Sterility not guaranteed if package is damaged or opened. Discard package if damaged. Avoid contact with eyes. Reuse will cause risk of infection and loss of efficacy.



STERILE R

MD Medical Device

REF 210

CE
2797

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP
The Hague
The Netherlands